# COMFORTISSE<sup>®</sup> POSTURE

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**[EN]** **Comfortisse<sup>®</sup> Posture** helps you achieve an upright posture effortlessly.

## Warnings:

- Wash **Comfortisse<sup>®</sup> Posture** before using it for the first time to reduce any odours. Odours will diminish over time if the product is kept well-ventilated.
- **Comfortisse<sup>®</sup> Posture** must fit tightly to the body, but without the

support becoming uncomfortable or the blood supply will be impaired. If you find **Comfortisse<sup>®</sup> Posture** uncomfortable loosen the fastener belts.

- Make sure you can breathe deeply while wearing **Comfortisse<sup>®</sup> Posture**.
- Do not use **Comfortisse<sup>®</sup> Posture** while sleeping.
- Stop wearing the garment if your skin breaks out or itches.
- Check the **Comfortisse<sup>®</sup> Posture** prior to each time of use for rips or damages, as the article must only be used in faultless condition.

## Product Description:

1. Belt
2. Elastic straps

## Instructions for Use:

1. Unfasten the belt.
2. Loosen the elastic straps.
3. Put your arms in (A).
4. Adjust the belt to make it as tight and comfortable as possible (B).
5. Fasten the belt end with the aid of the hook velcro fastener.
6. Adjust the elastic straps until you achieve a comfortable, upright posture (C).
7. Pass the ends of the elastic straps through the fasteners.

## Material Composition:

Lining: 100% polyester with a filling of styrene-butadiene rubber  
Elastic Belt: 44 % Elastodien, 30 % Nylon, 26 % Polyester  
Edging: 90 % Polyester, 10 % Elastan  
Belt area around Fastener: 100 % Polyester  
Fastener: 60 % Polyester, 40 % Nylon

## Cleaning instructions:

Do not expose to the sun | Handwash | Do not bleach | Do not dry in the tumble dryer | Do not iron | Do not dry-clean | Do not wring | Dry in the shade

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CH: Mediashop Schweiz AG | Leuholz 14 | 8855 Wangen | Switzerland
EU: MediaShop GmbH | Schneiderstraße 1, Top 1 | 2620 Neunkirchen | Austria