drive

DeVilbiss
HEALTHCARE

Rollator Cristallo

CE

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Indication / Contraindication

This walking aid is suitable for a single user with limited mobility. It is suitable for indoor and outdoor use.

The purpose of walking devices is to compensate for or at least improve restricted walking ability in users (expansion of range and/or mobility).

Persons with restricted hand/arm functions require special adjustments/versions of handgrips or hand rests/armrests.

For persons with balance disorders are walking frames not recommended, this is to be checked by an experiment in an individual case. If it is necessary a companion must be present.

Scope of delivery contents

1. Rollator Cristal
2. Basket
3. Tray
4. Cane Holder
5. Operation instructions

Installation

Height adjustment handgrips

Insert the tube of the adjustable handgrip into the frame at least to the "MAX" marking.

Fix it in the desired height with the fixing lever by turning it clockwise.

Adjust both handgrips on the same height / level.

⚠ Maximum allowable angle between the longitudinal axis of the handle and the driving direction is 0°

Fixing lever itself can be adjusted in the desired position by simultaneously pulling and twisting. For readjustment of the height of the handgrips unlock the fixing lever by turning it counterclockwise. Then fix the handgrip how described above.

Unfolding

Place the walking device on its wheels.


By pulling one of the pipes of a back wheel you can unfold the frame (1), then press the crossbar down (2) until it easily engages. Turn the movable folding protection down (3) until this stands vertically to the crossbar.

Fitting basket and tray


Hook in the basket in the hooks at the frame under the seat plate and place the tray with its small feet in the holes in the seat plate (see picture).

Breaking / Parking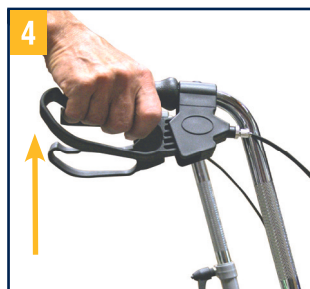

Pull both brake levers upwards to apply the brakes whilst the rollator is moving (picture 4). Always brake slowly and steady.

To lock the brakes, ensure the Rollator is stationary and press both brake levers down until they lock into position (picture 5).

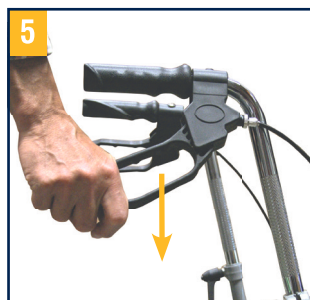To release the brakes from the locked position, pull up on the brake levers. (both brakes are in a middle position).

The effect of the brakes can be affected by abrasion of the wheels. Therefore regularly test the brakes.

Adjustment of brakes

For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture 6) or with the suitable tool (not included) on the brake wire (picture 7).


Sitting

- Position the rollator on a flat and solid surface.
- Then lock the brakes (parking position).
- The rollator must stand and must not slip.
- Sit straight up and do not lean backwards (see picture 8).

Folding / Transport / Storage

For folding open folding protection first (9) then pull seat upwards (10). Walking device can now be transported by trailing or is now suitable for storage (picture 11).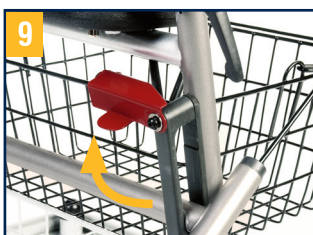

Safety instructions

- Use only as a walking aid!
- Use only on even and solid surfaces.
- The rollator must not be used for standing on or as a wheelchair! It must not be used for transport of persons or animals.
- Maximum user weight: 130 kg
- The use on escalator`s is expressly forbidden!
- Regularly check the screws and fasteners on the rollator and retighten if necessary.
- Use tray only for small, light, but not for pointed or hot objects. Max. load: 5.0 kg
- Use basket only for transport of objects and do not overload. Max. load: 5.0 kg
- Do not expose the product to temperatures above 40°C for a long time, as this can cause the rollator parts to overheat and burns to the user.
- The effect of the brakes can be affected by abrasion of the wheels. Therefore test regularly the brakes.

Cleaning / Maintenance

Clean regularly by hand with a damp cloth! Do not use any aggressive detergents!

Reprocessing

The product is suitable for reprocessing in line with manufacturer's recommendations.

Warranty

We issue a warranty of 24 months from date of purchase.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, handgrips, and brakes which are subject to normal wear and need periodic replacement.

Subject to technical alterations. Errors and omissions excepted.

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Distributor: DeVilbiss Healthcare GmbH • Kamenzer Straße 3 • D-68309 Mannheim • Germany • kontakt@devilbisshc.com •